

A-dec® Dental Furniture Installation Checklist

For more detailed information, see the installation guides for the dental furniture.

Model	S/N	Model	S/N
12 O'Clock Console:		X-Ray Storage Unit:	
Side Console:		Sterilization Center:	
Central Console:		Specialty Cabinet:	
Wall-Mounted Cabinet:			
Other:			

- Dental furniture is level, plumb, and securely anchored.
- Door and drawer faces are properly adjusted for alignment and uniform gaps.
- Foot pedals function properly.
- All task lights turn on/off when the rocker switch on the cabinet is pressed. Make sure all lights illuminate.
- X-ray/bifold storage doors operate smoothly and do not drag or chatter.
- The monitor rotates properly, moves up and down easily, and does not drift.
- A-dec ICV® functions properly.
- Control center functions properly, including the operation of the ICV, light, chair, delivery system, and auxiliaries.
- Date and time have been set on the control center.
- All power-assisted doors and drawers function properly.
- Dryer cycle activates and deactivates properly.
- All covers, caps, and shelves are in place.
- Clinical staff has been instructed how to access the Instructions for Use.

Doctor / Clinic:

Address:

Telephone:

Installation Date:

Dealer / Branch:

Installer:

Operatory/Room #:

Notes:

Doctor / customer unavailable to review checklist.

Reviewed with doctor / customer:

Signature

Date

 **A-dec Headquarters**

2601 Crestview Drive
Newberg, OR 97132 USA
Tel: 1.800.547.1883 Within USA/CAN
Tel: +1.503.538.7478 Outside USA/CAN
www.a-dec.com

A-dec, Inc. makes no warranty of any kind with regard to the content in this document including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.


a dec[®]
reliablecreativesolutions[®]



86.0624.00 Rev C
Date of Issue 2024-02-01
Copyright 2024 A-dec, Inc.
All rights reserved.
IGgphpor23