Porta-Chair

INSTRUCTIONS FOR USE

Setup

1. Turn the crank clockwise to raise the chair to the desired height. Figure 1 shows the chair in the folded down position.

NOTE The crank may be installed on either side of the chair. Remove the crank from the universal joint at the quick-disconnect. See Figure 2.

2. Use the connecting pin to connect the adjustable rod to the backrest assembly (see Figure 3). Adjust the chair backrest to the position desired. To raise, push the backrest forward; to lower, push the backrest adjusting lever in.

3. Install the toeboard to the chair by sliding the knobs into the slots on the chair frame, then tighten the knobs securely. See Figure 3.
4. To install the optional 25-inch post, insert the post in the mounting bracket on the side of the chair and tighten the thumbscrew.

Figure 1.

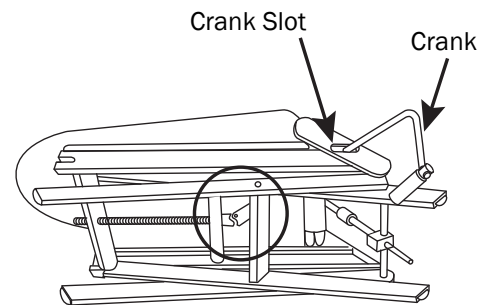
Figure 2.


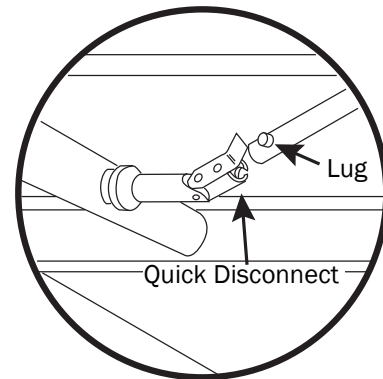
Figure 3.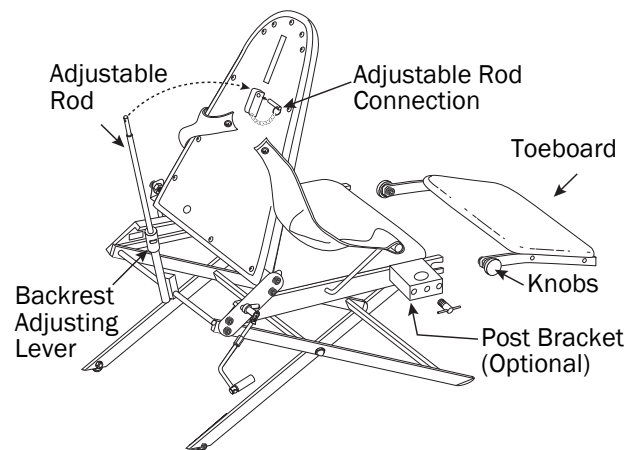

Breakdown

1. Remove the optional 25-inch post, if so equipped.
2. Loosen the knobs on the toeboard and remove the toeboard. Place the toeboard on the seat of the chair.
3. Pull the connecting pin from the adjustable rod. Lower the adjustable rod, then fold the chair back and the adjustable rod over the seat.
4. Turn the crank counter-clockwise until the chair is completely folded, as in Figure 1.
5. Place the chair in the optional case, if applicable, as shown in Figure 4. For comfort, the case may be carried as shown in Figure 5.

Cleaning the Chair

A mild soap or foam-type upholstery cleaner may be used on the vinyl. In the case of stubborn or embedded dirt in the grain of the vinyl, a fingernail brush or other soft-bristle brush may be used with the mild soap application.

All external metal surfaces may be cleaned using a detergent solution. Never use abrasive cleansers or scrubbing pads; they will damage the finishes.

Figure 4.


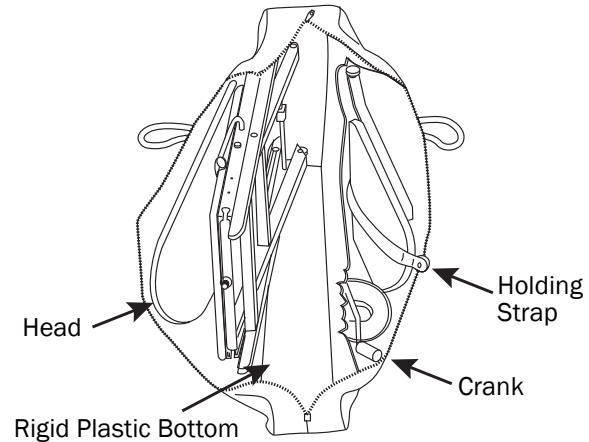
Figure 5.

Regulatory Information

Regulatory information mandated by agency requirements is provided in the *Regulatory Information, Specifications, and Warranty* document (p/n 86.0221.00), which is available at www.a-dec.com in the Document Library.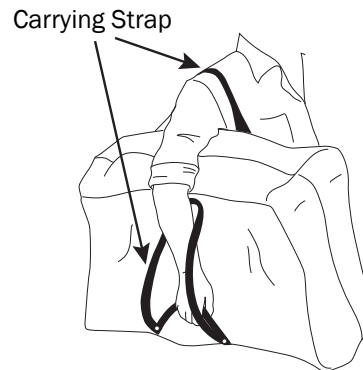

A-dec Headquarters

2601 Crestview Drive
Newberg, OR 97132 USA
Tel: 1.800.547.1883 Within USA/CAN
Tel: +1.503.538.7478 Outside USA/CAN
www.a-dec.com

A-dec Inc. makes no warranty of any kind with regard to the content in this document including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

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Copyright 2020 A-dec Inc.