


Recommended Monitor Mount Placement

The table below shows recommended locations to place the A-dec Ceiling Monitor Mount. Primary recommendations minimize light and monitor arm interference.

IF THE SYSTEM HAS THIS TYPE OF LIGHT:	INSTALL THE MONITOR MOUNT HERE:	
	Primary Recommendation	Secondary Recommendation
Track Mount Dental Light	C2*	S1 or S2
Ceiling Mount Dental Light	C1	S1 or S2
Chair Mount Dental Light	S1 or S2	C1
Cabinet or Wall Mounted Dental Light	S1 or S2	C1

* If installing at C2, position the monitor mount a minimum of 6-1/2" (165 mm) in front of the track light (towards 12 o'clock).

* NOTE: C2 position restricts comfortable use of monitor to only supine patient position.



A-dec Headquarters
 2601 Crestview Drive Newberg, Oregon 97132 USA

Tel: 1.800.547.1883 Within USA /Canada
 Tel: +1.503.538.7478 Outside USA /Canada
 www.a-dec.com

**A-dec 511ft/511B
 Dental Chair
 Mini Template**

86.0756.00 Rev D Copyright 2023 A-dec, Inc.
 Date of Issue 2023-10-23 All rights reserved.

