



Recommended Monitor Mount Placement

The table below shows recommended locations to place the A-dec Ceiling Monitor Mount. Primary recommendations minimize light and monitor arm interference.

| IF THE SYSTEM HAS THIS TYPE OF LIGHT: | INSTALL THE MONITOR MOUNT HERE: | |
|---------------------------------------|---------------------------------|--------------------------|
| | Primary Recommendation | Secondary Recommendation |
| Track-Mounted Ceiling Light | C2* | S1 or S2 |
| Fixed-Mount Ceiling Light | C1 | S1 or S2 |
| Chair Mount Light | S1 or S2 | C1 |
| Preference-Mounted Light | S1 or S2 | C1 |

* If installing at C2, position the monitor mount a minimum of 6-1/2" (165 mm) in front of the track light (towards 12 o'clock).

* NOTE: C2 position restricts comfortable use of monitor to only supine patient position.

A-dec Headquarters
 2601 Crestview Drive Newberg, Oregon 97132 USA
 Tel: 1.800.547.1883 Within USA /Canada
 Fax: 1.503.538.0276 Within USA /Canada
 Tel: +1.503.538.7478 Outside USA /Canada
 www.a-dec.com

**A-dec 511A
 Dental Chair
 Mini Template**

Copyright 2021 A-dec Inc.
 All rights reserved.
 86.0905.01 Rev G

